



Cover Cleaning Instructions:

Please note that the covers are designed to protect the products and should not be removed while the product is in use, as this will void the warranty.

For WipeN'Dry Covers:

- Routine Cleaning: Wipe clean with neutral soap suds and lukewarm water. Rinse and dry.
- Hard to Clean Areas: Use soft sponge with neutral soap suds and lukewarm water. Rinse and dry.
- Disinfection: Use properly diluted germicides or disinfectants. (Manufacturer's directions must be followed). As recommended by the Centers for Disease Control, a 1:10 dilution of household bleach.

For DryN'Cool Covers:

- Routine Cleaning: Use soft sponge with neutral soap suds and lukewarm water. Rinse and dry. Or Machine Wash in cold water, hang to dry.
- Hard to Clean Areas: Machine wash in cold water, hang to dry.
- Disinfection: Use properly diluted germicides or disinfectants. (Manufacturer's directions must be followed). Do not use bleach.

For Internal Covers:

- It is not recommended that the internal cover be removed, as it protects the cushion from being soiled.
- Routine Cleaning: Apply a soft liquid detergent and wipe dry.
- Hard to Clean/Disinfection: Machine wash in cold water and hang to dry. Do not allow the cushion to be used while the cover is off the cushion.

If you have any questions please do not hesitate to contact us at 289-597-8428.