

Drop Seats



Vision's drop seats are available for all of our cushions, including the VerySOFT, VIVE and Epic cushion lines in both standard and custom sizes and shapes.

- The drop seats are a universal fit and can accommodate any wheelchair
 - The drop clips self adjust to the chair's rail diameter, and do not require a locking mechanism to remain securely in place.
 - The drop clips are tension based and self adjust the chair's rail diameter ($\frac{3}{4}$ " to $1\frac{1}{8}$ ") without any additional devices/accessories.
 - The drop seat insert has a cut out to accommodate the cross-braces of the chair; as well the cushions have a bevelled edge that ensures easy installation.

To install Vision Drop-Seats:

- Place the hooks with the drop-clips on top of the wheelchair rails (the hooks are sent out loose so that the bolts are not tight and can be placed in the right location)
 - Ensure that there is no hardware that will be in the way
 - the drop clips can be attached at the base of the cushion at multiple points to ensure that they can be installed around the chair's existing hardware
- Press the drop-seat down (it is simplest to sit on it)
- After the clips grip the rail, turn the wheelchair on its back so that you have easy access to the bottom of the drop seat, tighten the bolts

To remove the Drop Seat:

- Tap the bottom of the drop seat firmly
 - at the front centre of the insert
- Lift the drop seat out of place.

If you have any questions please do not hesitate to contact us at 289-597-8428.